



BANDELETTES DE CONTRÔLE / CONTROL TEST STRIPS FOR ANIOS OXI'ZYME ANIOS OXI'ZYME HL

INDICATIONS

50 bandelettes de contrôle du taux d'acide peracétique dans la solution reconstituée d'ANIOS OXI'ZYME - ANIOS OXI'ZYME HL.

MODE D'EMPLOI

- Tremper la bandelette pendant une seconde, la sortir en la secouant pour éliminer l'excès de liquide.
- Lire le résultat 10 secondes après trempage. Ne jamais interpréter un résultat obtenu plus de 15 secondes après trempage.
- Pour la fréquence d'utilisation, se reporter au mode d'emploi de l'étiquette de l'ANIOS OXI'ZYME - ANIOS OXI'ZYME HL.

CONDITIONS DE STOCKAGE

- Refermer le tube après prélèvement d'une bandelette.
- Conserver à l'abri de l'humidité.
- Respecter la date limite d'utilisation reprise sur l'étiquette.
- Se conserve 90 jours après ouverture.
- Stocker entre +4°C et +30°C.

PRÉCAUTIONS D'EMPLOI

Respectez les précautions d'emploi. Dispositif médical de classe I. Avant toute utilisation, lisez l'étiquette et les informations concernant le produit.

INDICATIONS

50 test strips for the level of peracetic acid in the ANIOS OXI'ZYME - ANIOS OXI'ZYME HL reconstituted solution.

INSTRUCTIONS FOR USE

- Dip the strip for one second, remove it while shaking it to remove excess liquid.
- Read the result 10 seconds after dipping. Never interpret a result more than 15 seconds after dipping.
- For the frequency of use, refer to the instructions on the ANIOS OXI'ZYME - ANIOS OXI'ZYME HL label.

STORAGE CONDITIONS

- Close the container after removing each testing strip.
- Keep away from moisture.
- Respect the expiry date shown on the label.
- Store up to 90 days after opening.
- Store between +4°C and +30°C.

PRECAUTIONS FOR USE

Respect the precautions for use. Class I medical device. Before every use, read the label and information concerning the product.



NOIR UNIFORME: CONFORME
UNIFORM BLACK: COMPLIANT



**COLORATION NON UNIFORME, PRÉSENCE DE POINTS BLANCS,
ÉVANESCENCE RAPIDE DE LA COULEUR: NON CONFORME**
**NON-UNIFORM COLOURING, EVIDENCE OF WHITE SPOTS,
COLOUR SOON FADES: NON COMPLIANT**